


Wipe disinfection of small and medium-sized surfaces

Folding and unfolding a wipe correctly.



- only use the procedure shown here if the wipe size allows it
- always make sure the surface is completely covered with disinfectant solution
- when using ready-to-use wipes: after wipe removal, seal the packaging to prevent the remaining wipes from drying out



1. Put on gloves.

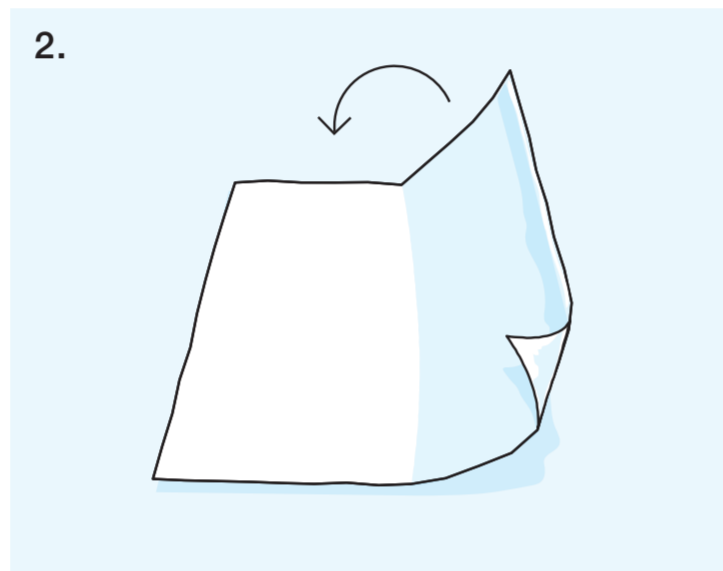


2. Fold the wipe in half twice.

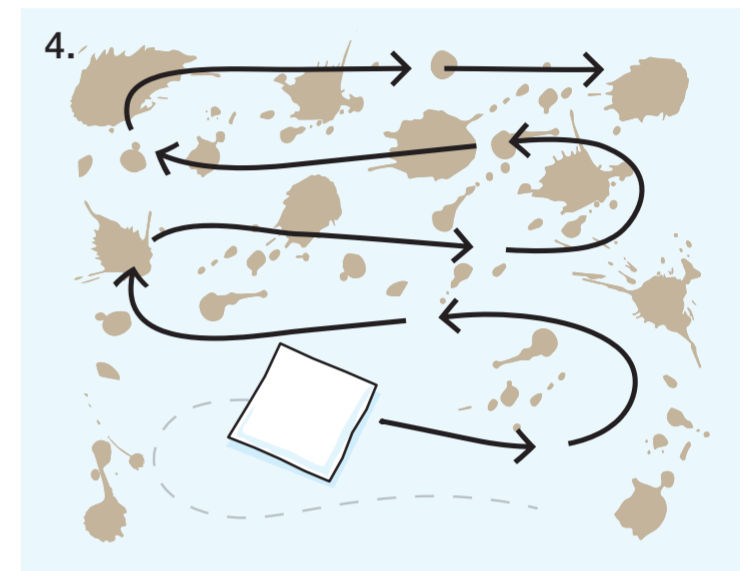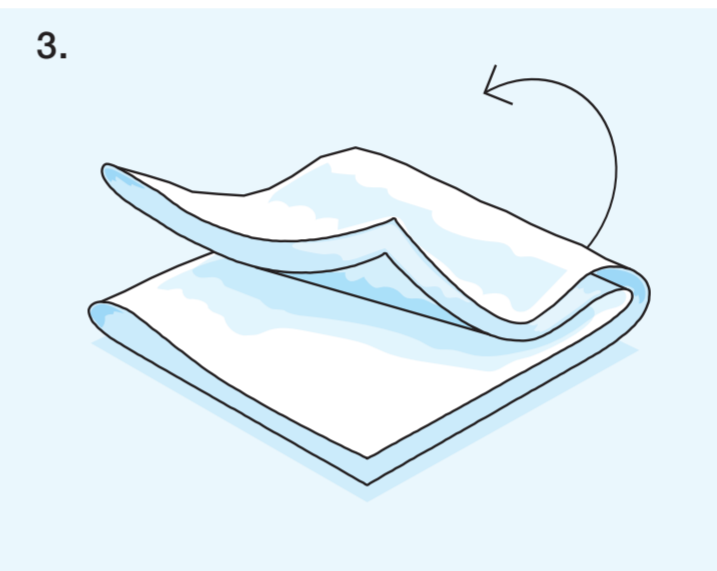


4. Wipe the surface thoroughly avoiding any gaps.




5. Flip the wipe over so that the unused side now faces the surface.

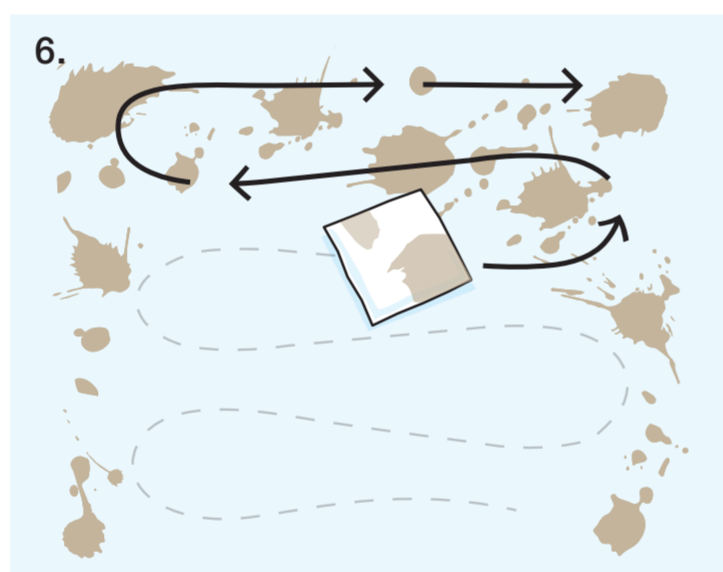


6. Proceed with the wipe disinfection of the surface.

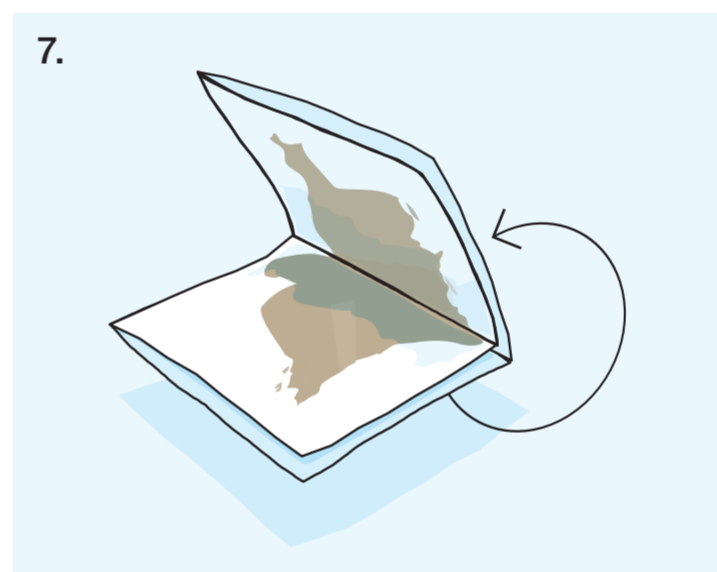



7. Unfold the wipe once and flip it over. Fold it in half again to bring the unused sides to the outside.

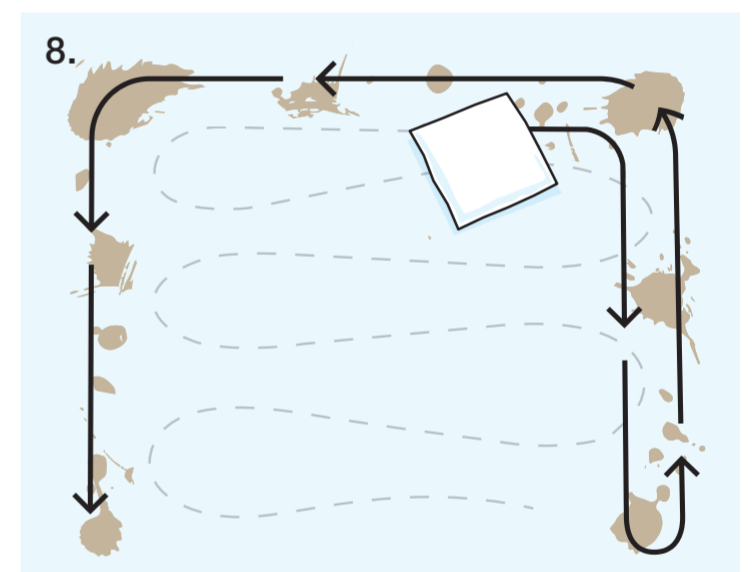

8. Thoroughly wipe the corners and edges. Flip over the wipe once more if necessary.

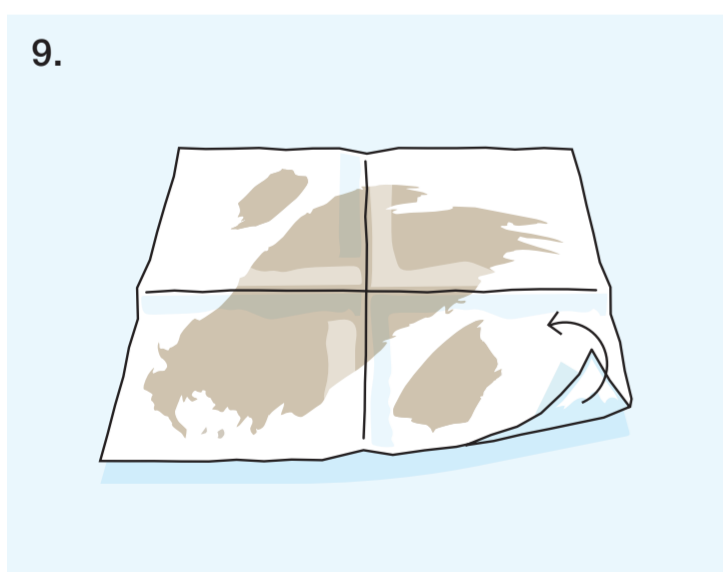

9. Unfold the wipe completely so that all used parts face upwards. Repeat the folding technique described in steps 2 and 3.

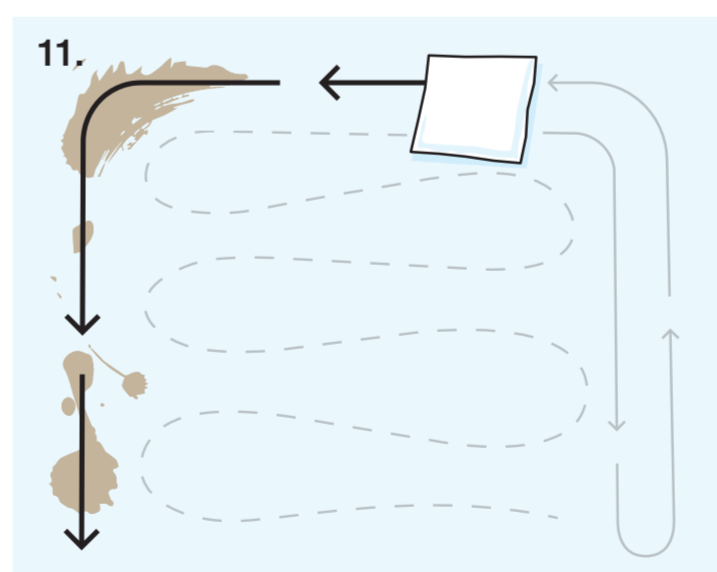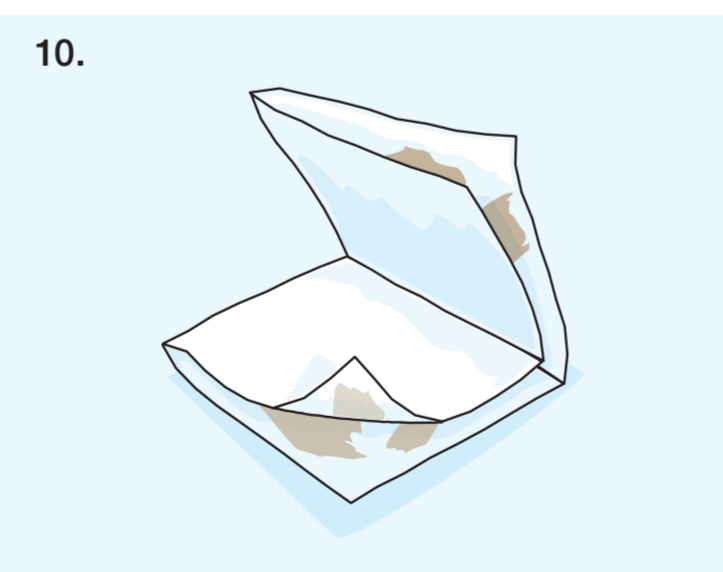

11. Proceed with the wipe disinfection of the remaining corners and edges.



12. After the surface disinfection, dispose of the wipe and gloves in accordance with local regulations.



Use surface disinfectants carefully. Always read the label and product information before use.



Find out more about our L&R surface disinfectants such as L&R surfacedisinfekt universal and L&R surfacedisinfekt alcohol.

<http://bit.ly/3LvYJ1v>



Visit our new platform L&R Prevent & Protect to find the answers to many more questions on infection control.

<http://bit.ly/3Nft3z1>